



## GUIDE TO KEEPING YOUR DEFIBRILLATOR **RESCUE READY**



Lucky2BHere provides devices manufactured by Cardiac Science. We currently have two models placed in communities across the country: 'G3' & 'G5'.



The following advice is applicable to both G3 and G5 Devices.

(If a different manufacturer supplied your device: [Contact Us](#) for tailored advice).

All L2BH devices and cabinets should be marked with L2BH Stickers, so there is a clear contact point for anyone concerned about the device. If your device requires stickers let us know: [Contact Us](#)

### **MONTHLY CHECKS**

These basic monthly checks will ensure the Device is **Rescue Ready** and able to save a life. We recommend that: **Check the device is indicating 'Rescue Ready'** is a **WEEKLY CHECK**.

1. **Check the device is indicating 'Rescue Ready'** (green within the Rescue Ready indicator). If the Rescue Ready Indicator is red with a black cross\* work through the rest of the checks to identify the cause.



2. **Open the temperature controlled external cabinet** – for the majority of L2BH cabinets this will trigger an alarm. The alarm is designed to draw attention to a potential emergency and will stop as soon as the cabinet is closed. It is usually worth warning anyone nearby that you will be setting off the alarm.



3. **Check that a Rescue Ready kit is in the cabinet**, this should contain:

- Reusable pocket mask
- Scissors
- Razor
- Pair of disposable gloves
- Paper towel



4. **Remove the Device** (in its carry case), from the external Cabinet

\* If the indicator is red with a black cross, make sure that you open up the Device during your checks. If the Device starts talking it's still working and it will still function in an emergency.

**L2BH Charity No: SC041103** All donations go towards continuing our work to place and maintain devices and provide Emergency Life Support Training: giving ordinary people skills and confidence to save a life - Thank You!



5. **Check the cabinet** for its integrity, and report any cracks or breaks in the structure: [Contact Us](#). Check for signs of water ingress/moisture and wipe away any moisture as required. If the air temperature is below 5 degrees, check that the cabinet feels warm. Close the cabinet (and enjoy the peace)!
6. **Check Pad expiry date** of the Pads connected to the device. This will be written in format: YYYY/MM/DD. If Pads are **one month** from expiry, see notes below and [Contact Us](#) to order new pads.
7. **Check battery indicator lights**. When the battery was brand new there were four green lights. Each green light signifies approx. 12-18 months of battery life. If one **red indicator light** is showing, see notes below and [Contact Us](#) to order a new battery.
8. If **Rescue Ready indicator was red with a black cross** and/or 'Maintenance Required' was displayed on the screen AND the Pads are in date AND the battery indicators are green OR on the very rare occasion that the screen is indicating an error code – **Call us: 07884 298079**

### Notes

**PADS** will usually be supplied with a 20-24 month expiry date. Pads will not stop working on the date of expiry and can be safely used in an emergency. We do however recommend that you **order new Pads one month before the expiry date**.

L2BH is very happy to support you with pad replacement. We usually send them out in sets of two (so you always have a spare set) and simply request a minimum donation of £60.00 towards our costs. Once replaced, please retain old pads and either give them back to a L2BH Trainer or send them to us, as they can be used for training.



**BATTERIES** have a lifespan of between 4-6 years. We recommend that you **only change batteries when the red indicator light shows**. If you act within the month of the red light showing, the device will work as normal.

L2BH is very happy to support you with battery replacement. For battery replacement we request a minimum donation of £200.00 towards our costs. Once replaced, drained batteries can usually be safely recycled at your local Recycling Centre - see your Council website for more details.



**Lucky2BHere Office**  
Bank of Scotland Building  
Somerled Square  
Portree, Isle of Skye IV51 9EH  
Tel: 01478 611522  
Email: [info@lucky2bhere.org](mailto:info@lucky2bhere.org)